

How to use the AirMini Universal Hose Adapter

The tube that arrives with your AirMini can only be used with selected ResMed masks. The Universal Hose Adapter for the AirMini allows you to use any CPAP mask with the device.

You will need a standard CPAP tube to use the adapter, if you do not have one consider the Universal Slimline 15mm Tube on [IntusHealthcare.com](https://www.intushealthcare.com).



STEP 1

Turn your AirMini onto its side to reveal the connection point.



STEP 2

Plug the adapter directly into the side of your ResMed AirMini.



STEP 3

Connect your CPAP tube to the open end of the adapter.



STEP 4

Connect your chosen mask to your CPAP tube.