

 **WARNING**
Risk of suffocation if full-face masks without an exhalation system are used!

If full-face masks without an exhalation system are used, the CO₂ concentration may rise to critical values and put the patient at risk.

- ⇒ Use full-face masks with an external exhalation system if there is no integrated exhalation system.
- ⇒ Follow the Instructions for Use for the exhalation system.

4. Connect the mask to the breathing tube (see Instructions for Use for the breathing mask).




The proper position and arrangement of the mask on the face of the patient is critical for uniform use of the device.

4.2 Start therapy

Requirement

Device is set up and connected (see "4.1 Set up and connect the device", page 13).

1. If the display is dark: press any key briefly.
The device switches to standby.
2. Briefly press the On/off key .


or

If the autoSTART function is activated: breathe into the mask.
Current therapy pressure appears in the display. Therapy starts.



For more information on autoSTART (see "5 Menu settings", page 19).

4.3 End therapy/switch off device

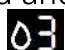
1. Briefly press the On/off key .
- or
- If the autoSTART function is activated: remove mask.
The device displays the therapy hours for the current day and then switches to standby.








To save energy, you can disconnect the plug from the socket during the day.

4.4 Set humidifier

Requirement

Humidifier is connected and filled with water (see Instructions for Use for humidifier).
The humidifier symbol  can be seen in the display.

1. Start therapy (see "4.2 Start therapy", page 15).
The humidifier switches on automatically. The humidifier symbol goes green .
2. To increase the humidifier stage: press the  key.
3. To decrease the humidifier stage: press the  key.
4. To switch off the humidifier: press the  key until  appears in the display.
 - The humidifier stage suitable for you depends on room temperature and humidity. If you have dry airways in the morning, heating output is set too low. If condensation has formed in the breathing tube in the morning, heating output is set too high.
 - When the water level in the humidifier is too low, the device switches off the humidifier automatically.
 - If the humidifier symbol is flashing, you need to fill the humidifier with water (see Instructions for Use for humidifier).










4.5 Performing the mask test

Requirement: Therapy is running.

1. Press menu button .



2. To start the mask test: Press .
The remaining time and the mask test pressure are displayed.
3. If necessary: Press  or  button to change the mask test pressure.
4. Check mask for leaks.
 - Proper mask location: Green  checkmark.
 - Mediocre mask location: Leakage display  turns orange.
 - Poor mask location: Leakage display  flashes.
5. If necessary: Adjust mask.
6. Wait until the device has completed the mask test.
or
Press .